



ROLLATOR
TRANSPORT CHAIR

INSTRUCTION GUIDE



**SAFE MOBILITY ASSISTANCE
FOR USER AND CAREGIVER**



**COMFORTABLE
TEMPORARY SEAT**



**300 POUND
WEIGHT CAPACITY**



EZ-FOLD DESIGN

**ALL INSTRUCTIONS ENCLOSED MUST
BE READ, UNDERSTOOD, BEFORE USE.**

*around town,
safely, in style*

COMFORTABLE TEMPORARY SEATING



live actively
Juvo[®]

CONTENTS:

Carefully remove the Transport Chair frame and parts from the box and inspect them for any damage or missing parts that may have resulted from shipping.

Call 1.844.277.6191 TOLLFREE with any questions.

PARTS INSIDE BOX:

- (1) Transport Chair
- (1) Box with Adjustable Footrests
- (1) Instruction Guide (Must be read **completely** before use)

ASSEMBLY INSTRUCTIONS:

1. Open the Transport Chair
 - Place the Transport Chair on the floor.
 - Push down on both sides of the seat frame until the seat is level (see image 1) and locked in place.
2. Backrest
 - Lift the backrest to upright position.
 - The lock to secure the backrest will automatically engage.
3. Leg Rests
 - Align holes on the leg rest arms with the pegs on the frame such that the footrest pads face inward (see image 2,3,4).
 - Swing into position until leg rest arms lock.
 - To adjust length of the leg rests, loosen the black lever on the back of the leg rest arm. Use the push button pins to adjust leg rests to desired length. Make sure pin is fully engaged in desired hole, retighten the same black lever (see image 4).
 - **Footrest pads arms swing up and out to make getting in and out of chair easier.**





OPERATING INSTRUCTIONS:

1. SEAT BELT

- Seat belt should worn by person being transported at all times (see image 5).



2. BRAKES: SITTING IN AND GETTING OUT OF THE CHAIR

- Brakes should always be engaged whenever someone sits in the seat or leaves the chair (see image 6).
- To engage the brakes, pull back on the brake lever until metal pad fully contacts the tire. (see image 6).
- **Caregiver assisting person in the chair should always stand behind the chair with hands on the handgrips to stabilize the chair when the person is getting in or out of the Transport Chair.**
- Brakes should be fully released (metal pads not touching the tire) when moving the Transport Chair.



3. FOLDING TRANSPORT CHAIR

- Unit can be folded for storage or transport by lifting straight up from middle of seat (see image 7).
- Backrest can also be lowered for more compact storage, reverse Step #2 Assembly Instructions (see Image 8).
- Leg rests can be removed, if desired, by reversing steps-see Assembly Instructions #3 Leg Rests previous page.



SAFETY PRECAUTIONS:

1. READ ALL ASSEMBLY, OPERATING INSTRUCTIONS BEFORE USE

2. Please make sure Transport Chair is fully opened, seat is down per #1 assembly instructions.
3. Make sure all knobs, nuts, bolts are fully tightened. Check regularly to ensure they are secure at all times.
4. Make sure the backrest is folded up into locked position.
5. Test brakes by depressing brake levers.
6. Always lock brakes before the person sits in or gets out of the chair.
7. Transport Chair is intended to be used by two people-the person needing transport and the caregiver pushing from behind.
8. Caregiver should always stand behind the chair holding the hand grips to steady the chair whenever the person sits in or gets out of the chair.
9. Replace tires when significantly worn.

**Call Juvo Products
Customer Service
Toll Free: 1.844.277.6191
with any questions**

**JUVO PRODUCTS, LLC
ROLLATOR LIMITED LIFETIME WARRANTY**

Juvo takes great pride in providing products of superior quality, design and function.

This Limited Lifetime Warranty applies to Rollator products manufactured by or on behalf of Juvo Products, LLC, (Juvo), under the brand name “Juvo®” and referred to in this Limited Lifetime Warranty as “Product.”

Product is warranted to be free from defects in materials and workmanship for so long as the original owner owns such Product. In the event of any such defect during the warranty period, Juvo® will remedy the defect as described herein, without charge to the purchaser, as soon as practicable. The remedy will consist of repair or replacement of the Product, or refund of the purchase price, at the discretion of Juvo® after opportunity to inspect the alleged defect or photographs thereof. All freight and labor charges will be the responsibility of the purchaser.

This Limited Lifetime Warranty applies to the original owner only and is non-transferable. This Limited Warranty does not cover:

- Labor for any inspection.
- Labor for replacement of any defective item(s).
- Incidental or consequential damages.
- Minor cosmetic defects, which do not affect performance or integrity of a part or the entire Product.
- Non-durable parts that are subject to normal wear and tear, such as Wheels, Hand Grips, Padding, Cables-Brakes, Upholstery, Bag, Plastic Parts.
- Vandalism, improper use, or negligence of the owner.
- Scratches or scrapes to materials or parts, dents, tears in fabric, or other issues that arise out of normal use of the Product.

This Limited Lifetime Warranty applies to Product purchased for personal use only. Product purchased for institutional multi-use environments are excluded from this warranty.

Juvo® Products have been designed for safety and quality. Any modifications made to the original Product could damage the structural integrity of the unit leading to failure and possible injury. Modification of the original Product voids any and all warranties and Juvo® assumes no liability for any modified Products or consequences resulting from failure of a modified Product.

Warranty Claims: The purchaser must notify Juvo directly of any claims under this Limited Lifetime Warranty within seven (7) days following the discovery of the problem. To make a claim under this Limited Lifetime Warranty, the Buyer should contact Juvo Products, LLC at:

**111 North Street, Suite 205
Barrington, Illinois 60010
Attention: Customer Service**

TOLL FREE CUSTOMER SERVICE: 844.277.6191

A copy of the original purchase receipt should accompany any claim under this Limited Lifetime Warranty.

Juvo shall have no obligations under this Limited Lifetime Warranty if the purchaser fails or refuses to reasonably cooperate in Juvo’s investigation of the claim (such as failing to provide photographs and or written documentation of the claim). If Juvo determines there are defects covered by this Limited Lifetime Warranty, Juvo will have up to thirty (30) days after receipt of notification to process the purchaser’s claim.

This Warranty gives you specific legal rights. You may have other rights as well which vary from state to state or province to province. This warranty excludes all consequential damages, however, some states do not allow the limitation or exclusion of consequential damages and therefore this limitation may not apply to you.



**SEE ALL JUVO LIFESTYLE PRODUCTS
AT JUVOPRODUCTS.COM**

Made in China



111 North Street, Suite 205
Barrington, Illinois 60010

1.844.277.6191 TOLL FREE

juvoproducts.com