



ROLLATOR

ROLLATOR-
TRANSPORT
CHAIR

INSTRUCTION GUIDE



COMPLETE BALANCE
SUPPORT



COMFORTABLE
TEMPORARY SEAT



HEIGHT ADJUSTABLE



250 POUND
WEIGHT CAPACITY

**ALL INSTRUCTIONS ENCLOSED MUST
BE READ, UNDERSTOOD, BEFORE USE.**

*walk safely.
walk in style.*

TWO PRODUCTS IN ONE



live actively
Juvo[®]




CONTENTS:

Carefully remove the Rollator-Transport Chair frame and parts from the box and inspect them for any damage or missing parts that may have resulted from shipping.

Call 1.844.277.6191 TOLLFREE with any questions.

PARTS INSIDE BOX:

- (1) Unit (backrest, wheels, seat already attached)
- (1) Carry bag
- (2) Foot rests (left and right)
- (1) Instruction Guide (Must be read **completely** before use)

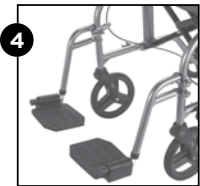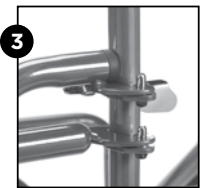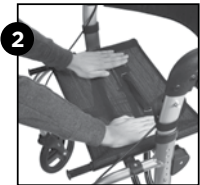

ASSEMBLY INSTRUCTIONS:

1. Place unit on the floor, seat facing up (see image 1). Push down on seat until it is locked in place (see image 2).
2. Place carry bag over plastic extensions at the front of the seat, with bag opening facing toward the front (see image 1).
3. **For Transport Chair mode:**
Attach footrests over extension pins above the front wheels and swing into position until latches click securely into place (see image 3,4). Fold footplates down into position.
4. Backrest should face back of unit, over back wheels, for Transport Chair mode (see image 5, 11).
5. Backrest should face forward, over front wheels, for Rollator mode (see image 1).

OPERATING INSTRUCTIONS:

1. TO OPEN ROLLATOR-TRANSPORT CHAIR

- Place unit upright on floor (see image 1)
- Push down on seat until it is locked in place (see image 6).

2. TO CLOSE ROLLATOR-TRANSPORT CHAIR

- Lift up on sewn handle in middle of seat until unit folds in (see image 7).

3. TO ADJUST HANDLE HEIGHT:




- User should stand behind unit with arms hanging at their side and feet lined up with rear wheels (see image 8).
- Handles should be even with wrists for proper fit.
- Use push pins on sides of handle uprights to adjust height accordingly. Make sure push pins are fully engaged in holes and handles are same height (see image 9).

4. ROLLATOR MODE

- Backrest should be flipped to front of unit over front wheels (see image 1).
- Footrests can be removed (see Assembly Instructions) or folded back and clipped to side of unit. Flip front footrests up. Unclip latches near extension pins by pulling away from the unit to release the footrest. Swing footrests back on side of unit. Click footrests into plastic clamps to hold into place (see image 10).

5. TRANSPORT CHAIR MODE


- Flip backrest so that it faces back of unit, over back wheel (see image 5, 11).
- Make sure footrests are in correct position (see image 12, next page) Refer to Assembly Instructions #3.

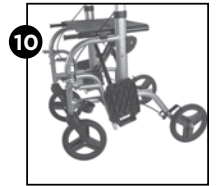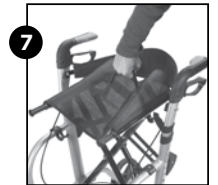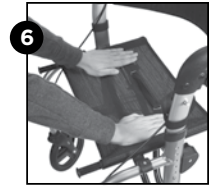



6. BRAKES

- Squeeze the brakes to stop the Rollator-Transport Chair (see image 13).
- Push the brakes straight down to the locking position (see image 14).
- BRAKES MUST BE IN THE LOCKED POSITION WHENEVER SITTING ON THE ROLLATOR-TRANSPORT CHAIR. ALWAYS ENGAGE, DISENGAGE BRAKES SIMULTANEOUSLY.

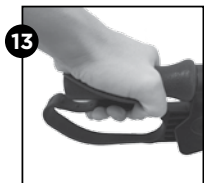

4. CHECK HAND BRAKES


- Hand Brakes should be in working order after assembly.
- Check Hand Brakes periodically for proper function.
- Squeeze the hand brakes and check to ensure that the metal brake arms engage the wheels and stop the Rollator-Transport Chair (see Image 15).
- Push the Hand Brakes DOWN into their locking position and check to ensure that the metal brake arms engage the wheels and stop the Rollator-Transport Chair (see Image 15).

5. HAND BRAKE ADJUSTMENT

- Hand Brakes should be in working order after assembly.
- However brakes can loosen during normal use.

TIGHTEN BRAKES

- Place Rollator-Transport-Chair upright on floor.
- Loosen nut at top of brake shoe assembly, near back wheel. Turn **counter-clockwise** until desired tension is achieved (see image 16).
- Test by squeezing hand brake — ensure that metal brake shoe fully engages the wheel.
- Adjust as needed.

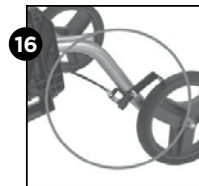

LOOSEN BRAKES

- Place Rollator-Transport Chair upright on floor.
- Loosen nut at top of brake shoe assembly, near wheel. Turn **clockwise** until desired tension is achieved (see image 16).
- Test by squeezing hand brake — ensure that metal brake shoe fully engages the wheel.
- Adjust as needed.

SAFETY PRECAUTIONS:

1. READ ALL ASSEMBLY, OPERATING INSTRUCTIONS BEFORE USE
2. Please make sure Rollator-Transport Chair is fully opened, seat is pushed down all the way (see image 2).
3. Make sure all knobs, nuts, bolts are fully tightened. Check regularly to ensure they are secure at all times
4. Make sure Rollator-Transport Chair is level, handles are level as per Operating Instructions #3.
5. Test Rollator-Transport Chair for stability before applying full weight to hand grips or seat.
6. Test brakes before using
7. ALWAYS LOCK BRAKES BEFORE SITTING, please refer to Operating Instructions, #6 Brakes and Image 14
8. Replace tires when significantly worn or damaged
9. Caution: Keep hands clear of frame when folding
10. Maximum weight capacity 250 lbs.

Call Juvo Products
Customer Service
Toll Free: 1.844.277.6191
with any questions

JUVO PRODUCTS, LLC ROLLATOR LIMITED LIFETIME WARRANTY

Juvo takes great pride in providing products of superior quality, design and function.

This Limited Lifetime Warranty applies to Rollator products manufactured by or on behalf of Juvo Products, LLC, (Juvo), under the brand name "Juvo®" and referred to in this Limited Lifetime Warranty as "Product."

Product is warranted to be free from defects in materials and workmanship for so long as the original owner owns such Product. In the event of any such defect during the warranty period, Juvo® will remedy the defect as described herein, without charge to the purchaser, as soon as practicable. The remedy will consist of repair or replacement of the Product, or refund of the purchase price, at the discretion of Juvo® after opportunity to inspect the alleged defect or photographs thereof. All freight and labor charges will be the responsibility of the purchaser.

This Limited Lifetime Warranty applies to the original owner only and is non-transferable. This Limited Warranty does not cover:

- Labor for any inspection.
- Labor for replacement of any defective item(s).
- Incidental or consequential damages.
- Minor cosmetic defects, which do not affect performance or integrity of a part or the entire Product.
- Non-durable parts that are subject to normal wear and tear, such as Wheels, Hand Grips, Padding, Cables-Brakes, Upholstery, Bag, Plastic Parts.
- Vandalism, improper use, or negligence of the owner.
- Scratches or scrapes to materials or parts, dents, tears in fabric, or other issues that arise out of normal use of the Product.

This Limited Lifetime Warranty applies to Product purchased for personal use only. Product purchased for institutional multi-use environments are excluded from this warranty.

Juvo® Products have been designed for safety and quality. Any modifications made to the original Product could damage the structural integrity of the unit leading to failure and possible injury. Modification of the original Product voids any and all warranties and Juvo® assumes no liability for any modified Products or consequences resulting from failure of a modified Product.

Warranty Claims: The purchaser must notify Juvo directly of any claims under this Limited Lifetime Warranty within seven (7) days following the discovery of the problem. To make a claim under this Limited Lifetime Warranty, the Buyer should contact Juvo Products, LLC at:

**111 North Street, Suite 205
Barrington, Illinois 60010
Attention: Customer Service**

TOLL FREE CUSTOMER SERVICE: 844.277.6191

A copy of the original purchase receipt should accompany any claim under this Limited Lifetime Warranty.

Juvo shall have no obligations under this Limited Lifetime Warranty if the purchaser fails or refuses to reasonably cooperate in Juvo's investigation of the claim (such as failing to provide photographs and or written documentation of the claim). If Juvo determines there are defects covered by this Limited Lifetime Warranty, Juvo will have up to thirty (30) days after receipt of notification to process the purchaser's claim.

This Warranty gives you specific legal rights. You may have other rights as well which vary from state to state or province to province. This warranty excludes all consequential damages, however, some states do not allow the limitation or exclusion of consequential damages and therefore this limitation may not apply to you.



LIFESTYLE PRODUCTS

that improve mobility and safety
around the home



BALANCE & MOBILITY



OPENING



DRESSING



REACHING



SAFETY



TOILETING



BATHING



CARRYING



SURGERY RECOVERY



DAILY LIVING AIDS



**SEE ALL JUVO LIFESTYLE PRODUCTS
AT JUVOPRODUCTS.COM**

Made in China



111 North Street, Suite 205
Barrington, Illinois 60010

1.844.277.6191 TOLL FREE

juvoproducts.com