

Self-Assist Toilet Aid

PURPOSE:

This product is specifically designed to provide assistance in the toileting, wiping function.


This product was designed by Occupational Therapists and is useful for people with limited mobility as result of injury, illness, medical condition.

This product is designed to be used from the front or back depending on the users specific condition.

Please consult with Occupational Therapist or Physician before use.

1 TISSUE OR WIPES

Gather 2-3 sheets of tissue or 1 wipe into a ball.

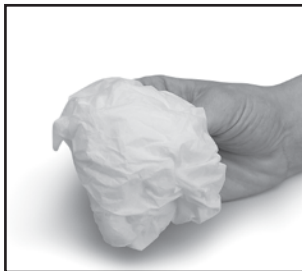


2 LOAD TISSUE/WIPES

Place tissue or wipe between two halves of tip — so that most of the tissue sticks out of the tip as shown. **Do not overload. A few sheets is sufficient**



3 LOAD TISSUE/WIPES

Squeeze up on trigger to hold tissue in place. Squeeze trigger to lock tip with tissue or wipe in place if desired.

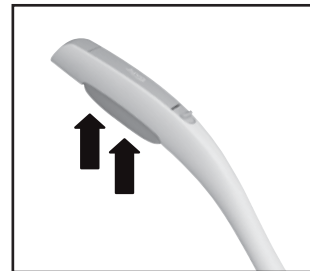

4 USE FROM FRONT

Use toilet aid between legs while sitting on toilet seat to wipe.




USE FROM BACK

Toilet aid may be used from back if able to squat over toilet. **Consult Occupational Therapist or Physician before using.**



5 RELEASE SOILED TISSUE

Push release button forward from either of two positions to release tissue. Repeat steps 1-5 as needed.

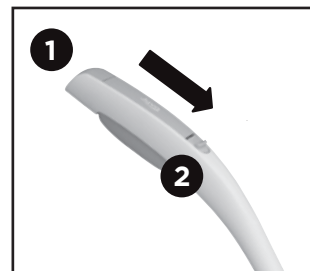

6 STORAGE

Clean tips with warm soapy water after use. This product is designed to sit on a flat surface without the tips touching the surface.



Scan to View YouTube Instructional Video

INSTRUCTIONS:

Self-Assist Toilet Aid with Caddy

PURPOSE:


This product is specifically designed to provide assistance in the toileting, wiping function.

This product was designed by Occupational Therapists and is useful for people with limited mobility as result of injury, illness, medical condition.

This product is designed to be used from the front or back depending on the users specific condition.

Please consult with Occupational Therapist or Physician before use.


1 TISSUE OR WIPES
Gather 2-3 sheets of tissue or 1 wipe into a ball.

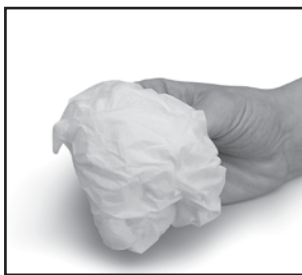

2 LOAD TISSUE/WIPES
Place tissue or wipe between two halves of tip — so that most of the tissue sticks out of the tip as shown. **Do not overload. A few sheets is sufficient**



3 LOAD TISSUE/WIPES
Squeeze up on trigger to hold tissue in place. Squeeze trigger to lock tip with tissue or wipe in place if desired.

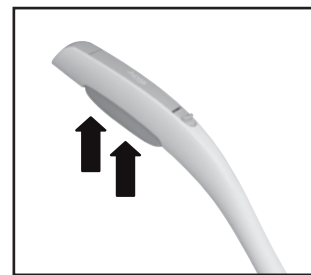


4 USE FROM FRONT
Use toilet aid between legs while sitting on toilet seat to wipe.



USE FROM BACK
Toilet aid may be used from back if able to squat over toilet. **Consult Occupational Therapist or Physician before using.**




5 RELEASE SOILED TISSUE
Push release button forward from either of two positions to release tissue. Repeat steps 1-5 as needed.

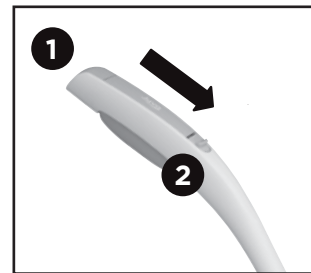

6 STORAGE
Clean tips with warm soapy water after use.

Squeeze up on trigger handle to lock tips in place.

Insert Toilet Aid into Caddy so that front of Toilet Aid is facing the taller side of the caddy.

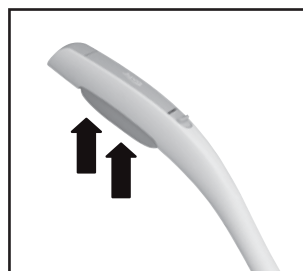

Scan to View YouTube Instructional Video

