

COMFORT SERIES™ Bath Safety Chair

Model: BSC01

CONTENTS

Carefully remove contents from box and inspect them for any damage that may have resulted from shipping, or missing parts.

Parts Inside Box:

- Qty. 1 Comfort Series Bath Safety Chair Seat with Foam Padding
- Qty. 1 Padded Back-Rest
- Qty. 2 U-Shaped Leg Braces. They are numbered 1 and 2
- Qty. 2 Back-Rest Support Bars
- Qty. 4 Adjustable Legs with Rubber Feet
- Qty. 4 Butterfly Bolts with Washers - Long
- Qty. 1 Butterfly Bolt with Washer - Medium
- Qty. 2 Butterfly Bolts with Washers - Short




1

IMPORTANT NOTE: The two U-Shaped Leg Braces are slightly different. It is important that you follow the instructions and use the correct U-Shaped Leg Brace in proper sequence. The U-Shaped Leg Braces are numbered with a sticker for this purpose.

ASSEMBLY INSTRUCTIONS

1. Turn Seat Upside Down

- Place seat on flat surface i.e. table or floor, upside down so underside is facing you

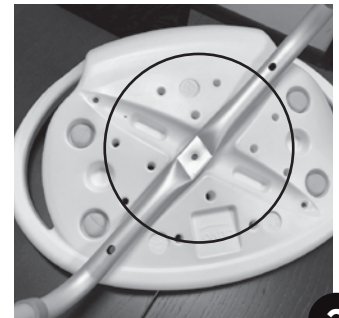


INCORRECT

2

2. Connect First U-Shaped Leg Brace

- Lay U-Shaped Leg Brace **#1** across underside of seat. Make sure it is resting in recessed channel, marked **#1**, such that the flat portion of the tubing is aligned with the direction of the channel. **IMPORTANT.** (Refer to image 2 and 3)

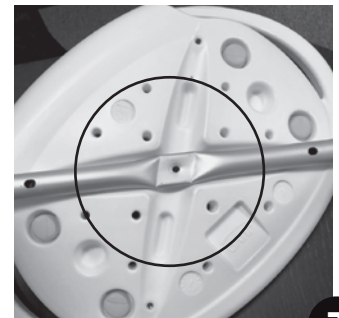

CORRECT

3

3. Connect Second U-Shaped Leg Brace

- Lay U-Shaped Leg Brace **#2** on top of, across U-Shaped Leg Brace **#1**, in recessed channel marked **#2**. (Refer to image 4)
- Fasten U-Shaped Leg Braces **#1** and **#2** to underside of seat with Butterfly Bolt with Washer **starting in the center first**. Use Butterfly Bolt with Washer - Medium to fasten center position first. Do not over tighten

4. Connect Back-Rest Support Bars and U-Shaped Leg Braces to Underside of Seat

- Lay one Back-Rest Support Bar across the U-Shaped Leg Braces such that the two holes of the Back-Rest Support Bar align with the two holes of the U-Shaped Leg Braces, (refer to image 5). Make sure that the Back-Rest Support Bar is facing up, from the back side of the chair. **IMPORTANT**
- Fasten Back-Rest Support Bar and U-Shaped Leg Brace to underside of chair with two Butterfly Bolts with Washers - Long. (Refer to image 6). Do not overtighten.
- Repeat process of fastening Back-Rest Support Bar and U-Shaped Leg Brace to underside of chair - with second Back-Rest Support Bar. (Refer to image 7).



4

5. Connect Padded Back Rest to Back-Rest Support Bars

- Turn Chair right side up.
- Insert two Back-Rest Support Bars into Padded Back Rest channels. (Refer to image 8).
- Fasten Padded Back Rest to Back-Rest Support Bars with two Butterfly Bolts with Washers – Short. (Refer to image 9).



5

6. Tighten All Butterfly Bolts with Washers

- Make sure all seven Butterfly Bolts with Washers are firmly fastened. Do not overtighten.



6

FEATURES: COMFORT SERIES BATH SAFETY CHAIR

- Integrated Handles to assist in standing and sitting
- Height adjustable legs from 14" – 21" for proper fit
- Soft padded cover for comfort
- Padded cover pops in and out for easy cleaning
- Seat and pad have drainage holes to shed water
- 300 lb. weight capacity
- No-tools easy assembly
- Fits most standard bathtubs, showers

SAFETY PRECAUTIONS

1. READ ALL INSTRUCTIONS BEFORE USE
2. Make sure all four Adjustable Legs with Rubber Feet are securely connected to U-Shaped leg braces at the same height adjustable position. Each push pin must click into place, be fully visible through the preferred height adjustable hole.
3. Make sure all fastening positions with Butterfly Bolts with Washers are firmly fastened. Do not overtighten
4. Make sure all four rubber feet are flat on the inside floor of the bathtub or shower
5. Test Comfort Series Bath Safety Chair for stability before applying full weight



7

JUVO PRODUCTS, LLC LIMITED LIFETIME WARRANTY

Juvo takes great pride in providing products of superior quality, design and function.

This Limited Lifetime Warranty applies to products manufactured by or on behalf of Juvo Products, LLC, (Juvo), under the brand name "Juvo®" and referred to in this Limited Lifetime Warranty as "Product."

Product is warranted to be free from defects in materials and workmanship for so long as the original owner owns such Product. In the event of any such defect during the warranty period, Juvo® will remedy the defect as described herein, without charge to the purchaser, as soon as practicable. The remedy will consist of repair or replacement of the Product, or refund of the purchase price, at the discretion of Juvo® after opportunity to inspect the alleged defect or photographs thereof. All freight and labor charges will be the responsibility of the purchaser.



8

This Limited Lifetime Warranty applies to the original owner only and is non-transferable. This Limited Warranty does not cover:

- Labor for any inspection.
- Labor for replacement of any defective item(s).
- Incidental or consequential damages.
- Minor cosmetic defects, which do not affect performance or integrity of a part or the entire Product.
- Non-durable parts that are subject to normal wear and tear, such as Wheels, Hand Grips, Padding, Cables-Brakes, Upholstery, Bag, Plastic Parts.
- Vandalism, improper use, or negligence of the owner.
- Scratches or scrapes to materials or parts, dents, tears in fabric, or other issues that arise out of normal use of the Product.

This Limited Lifetime Warranty applies to Product purchased for personal use only. Product purchased for institutional multi-use environments are excluded from this warranty.

Juvo® Products have been designed for safety and quality. Any modifications made to the original Product could damage the structural integrity of the unit leading to failure and possible injury. Modification of the original Product voids any and all warranties and Juvo® assumes no liability for any modified Products or consequences resulting from failure of a modified Product.